+++
The Original Leader,
Now Even Better

Biotrin Parvovirus B19 IgM EIA

+ HELLO!

At Biotrin, we have always been proud in the knowledge that we offer the best possible platform for diagnosis of Parvovirus B19 antibodies – our Biotrin Parvovirus B19 ELISA IgG and IgM assays.

Today we want to give you even more!

We are pleased to announce carefully selected additions and validations to key parameters of our kit to make it even more complete, more reliable and more user-friendly!

50% more controls!

- + Positive Control volume increased to 3ml (see page 8).
- + Negative control volume increased to 3ml (see page 8).
- + Calibrator (Cut-off Control) volume increased to 3ml (see page 8).

New validations for ease of use!

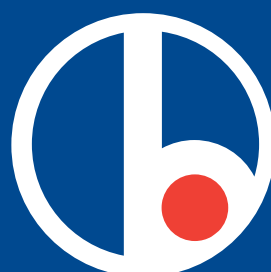
- + The Reconstituted Wash Solution is now stable for 2 weeks at room temperature.
- + Incubation can now be timed at ±5 minutes rather than ±2 minutes. This relates to steps 7, 11, and 15 of the assay procedure (see page 13-14).
- + The Optical Density no longer needs to be read immediately at the end of the assay – it can now be read any time within 30 minutes of adding the stop solution with an ELISA plate reader (step 20 of the assay procedure – see page 14).

Complete colour-coding!

The Biotrin VP2 Diluent cap has been changed to a white cap – this is the same as the VP2 conjugate (white cap) and avoids confusion with the positive control (red cap). (See kit components and page 8).

- + New precaution phrase added: "As the conjugate is light sensitive, prolonged exposure to light should be avoided" (see page 7).
- + New "Additional Materials Required" item added: 37°C incubator (see page 9).
- + New version of the Instructions for Use Booklet. Code becomes PEMV-358-03/07/07 (see back cover of booklet).

*For any questions regarding these new features, please contact your Biotrin representative.

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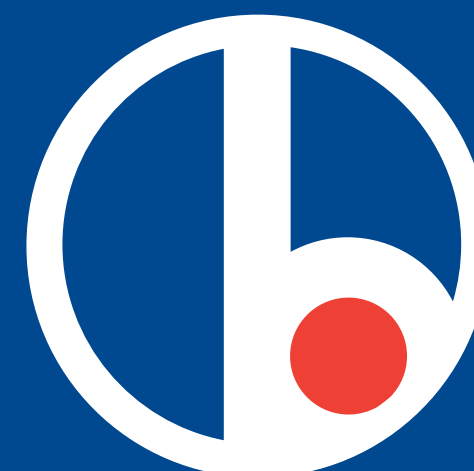
HELLO!

At Biotrin, we have always been proud in the knowledge that we offer the best possible platform for diagnosis of Parvovirus B19 antibodies – our Biotrin Parvovirus B19 ELISA IgG and IgM assays. Today we want to give you even more! We are pleased to announce carefully selected additions and validations to key parameters of our kit to make it even more complete, more reliable and more user-friendly! We hope that you will find these new features to be of value. Thank you once again for choosing Biotrin!

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